

COVID-19 Vaccine Ancillary Supply Kit Guidance

Purpose:

This guidance provides an overview of the COVID-19 vaccine ancillary supply kits the U.S. Department of Health and Human Services (HHS) is providing to enrolled COVID-19 vaccination providers as part of the federal COVID-19 Vaccination Program. It includes a general description of different COVID-19 vaccine ancillary kit configurations for administration, as well as a list of select corresponding products, product descriptions, and product quantities. For reference and to access additional product-specific information and training resources, this document also includes website hyperlinks and contact information for select product manufacturers.

Background:

HHS is providing ancillary supply kits for the administration of COVID-19 vaccine. The Strategic National Stockpile (SNS), managed by the HHS Office of the Assistant Secretary for Preparedness and Response (ASPR), is partnering with McKesson Corporation to produce, store, and distribute these vaccine ancillary supply kits on behalf of the SNS.

How to use this guide:

Jurisdiction immunization program staff—use this guide to help you:

- Learn about products that will arrive in ancillary kit(s).

COVID-19 vaccination providers—use this guide to help you:

- Learn about products that will arrive in ancillary kit(s).
- Prepare storage space for the ancillary kit products you will receive.

Ancillary kit basics:

How does a provider order ancillary kits?

You do not need to order ancillary kits. When you order COVID-19 vaccine in VTrckS, ancillary supplies will automatically be ordered in amounts to match the vaccine orders.

Note: To receive COVID-19 vaccine, vaccination provider facilities and organizations must enroll in the federal COVID-19 Vaccination Program coordinated through their jurisdiction's immunization program.

How much do ancillary kits cost?

Ancillary kits will be provided at no cost to enrolled COVID-19 vaccination providers.

How will needles and syringes be packaged?

Needles and syringes for vaccine administration may be packaged as integrated units (i.e., combo needles and syringes for which NO assembly required) or as separate items in a kit (i.e., assembly required).

Can I order specific brand(s) of needles and/or syringes?

Due to a limited supply of needles and syringes, specification of preferences for needles or syringes is not feasible. Products included in the kits may vary over time. In order to meet the demand for supplies, the federal government has purchased single-use, sterile needles and syringes from multiple manufacturers (to include foreign sources) to ensure adequate supplies. These products are approved by the U.S. Food and Drug Administration, safety-engineered and compliant with standards established by the Occupational Safety and Health Administration.

What is considered “pediatric” and “adult”?

For the purpose of immunizations pediatric is birth through 18 years and adult is 19 years and older. Refer to each product's EUA for age indications.

*The number of doses is limited by the dose volume extracted (i.e., 0.5 mL or 0.25 mL) and the number of times the vial stopper is punctured. Do not puncture the vial more than 20 times. Discard the vial after 20 punctures, including any remaining vaccine.
*Currently unavailable for order (8/20/2021)

COVID-19 Vaccine

Product Information Guide



COVID-19 Vaccine Ancillary Supply Kit Guidance (*continued*)

Related guidance and resources:

- COVID-19 Vaccination Program Interim Playbook for Jurisdiction Operations - PDF

Contact for questions:

Vaccination providers should contact the manufacturer with questions related to proper product use.

Becton Dickinson (BD) Worldwide	https://www.bd.com/en-us/	844-823-5433
Cardinal Health	https://www.cardinalhealth.com/en.html	1-800-964-5227
HTL Strefa	https://htl-strefa.com/home-page/	877-660-1900
Duopross Meditech Corp	https://www.duopross.com/	1-800-844-1350
Retractable Technologies Inc. (RTI)	https://retractable.com/	888-703-1010
Marathon/Smiths Medical	N/A	941-704-7864

For any issues with equipment (e.g., faulty equipment), contact McKesson Customer Service Team at 833-272-6634 or SNSSupport@McKesson.com.

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccines Supporting 100 Doses (Janssen)

The following tables list the contents of each possible ancillary kit by product and quantity. All kits are configured for 100 doses with 5% surplus.

Needle and Syringe Sizes for **Adult** Ancillary Kits

PRODUCT	QUANTITY
Needle (22–25G x 1")	85
Needle (22–25G x 1.5")	20
Syringe (1 mL or 3 mL)	105
Alcohol Pad (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	2
Surgical Mask	4

Needle and Syringe Sizes for **Pediatric** Ancillary Kits

PRODUCT	QUANTITY
Needle (25G x 1")	105
Syringe (1 mL or 3 mL)	105
Alcohol Pad (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	2
Surgical Mask	4

Needle and Syringe Sizes for **Mixed (Pediatric/Adult)** Ancillary Kits

PRODUCT	QUANTITY
Adult/Pediatric Needle (22–25G x 1")	95
Adult Needle (22–25G x 1.5")	10
Syringe (1 mL or 3 mL)	105
Alcohol Pad (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	2
Surgical Mask	4

Needle Gauge and Length Chart: www.cdc.gov/vaccines/hcp/admin/downloads/vaccine-administration-needle-length.pdf

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits: COVID-19 Vaccine Supporting 100 primary series doses or 200 booster doses (Moderna)

The following tables list the contents of each possible ancillary kit by product and quantity. Two supply kits will be provided with each minimum order size of Moderna. All kits are configured for 100 primary series doses or 200 booster doses with 5% surplus (i.e., two supply kits will support a maximum of 200 booster doses).

Needle and Syringe Sizes for **Adult** Ancillary Kits

PRODUCT	QUANTITY
Needle (25G x 1")	85
Needle (25G x 1.5")	20
Syringe (1 mL)	105
Alcohol Pad (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	2
Surgical Mask	4

Needle and Syringe Sizes for **Pediatric** Ancillary Kits

PRODUCT	QUANTITY
Needle (25G x 1")	105
Syringe (1 mL)	105
Alcohol pads (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	2
Surgical Mask	4

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccine Supporting 140 Doses (Moderna)

Ancillary kits included with Moderna 14 vaccine orders will include*:

- One 140-dose ancillary kit, with standard syringes (3 mL and/or 1 mL)
- One 100-dose ancillary kit, with only small syringes (1mL)

This approach will support administration of booster doses and primary series doses using the Moderna 14 carton. There will continue to be surplus ancillary supplies because each vial can only be safely punctured 20 times.

Needle and Syringe Sizes for **Adult** Ancillary Kits

PRODUCT	QUANTITY
Needle (22–25G x 1")	75
Syringe (1 mL or 3mL, LDV)	75
Needle (22–25G x 1")	50
Syringe (1 mL or 3mL)	50
Needle (22–25G x 1.5")	25
Syringe (1 mL or 3 mL)	25
Alcohol Pad (sterile, individually sealed)	300
Vaccination Record Card	150
Needle Gauge and Length Chart	1
Face Shield	3
Surgical Mask	6

Needle and Syringe Sizes for **Pediatric** Ancillary Kits

PRODUCT	QUANTITY
Needle (25G x 1")	75
Syringe (1 mL or 3 mL, LDV)	75
Needle (25G x 1")	75
Syringe (1 mL or 3 mL)	75
Alcohol pads (sterile, individually sealed)	300
Vaccination Record Card	150
Needle Gauge and Length Chart	1
Face Shield	3
Surgical Masks	6
Face Shield	3
Surgical Mask	6

Needle and Syringe Sizes for **Mixed (Pediatric/Adult)** Ancillary Kits

Kit contents not yet available

Note: Kit contents for mixed (pediatric/adult) kits currently unavailable.

*Beginning for orders approved Saturday, October 30, 2021

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccine Supporting 100 Pediatric Doses (Pfizer)

*Needle and Syringe Sizes for **Pediatric** Ancillary Kits*

PRODUCT	QUANTITY
Needle (25G x 1")	70
Syringe (1 mL, LDV)	70
Needle (25G x 1")	35
Syringe (1 mL)	35
Needles, Mixing (21–25G x 1.5")	20
Syringe, Mixing (3 mL or 5 mL)	20
Alcohol Pad (sterile, individually sealed)	210
Vaccination Record Card	100
Needle Gauge and Length Chart	1
Face Shield	3
Surgical Mask	6
Diluent, 10 mL	10

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccine Supporting 300 Pediatric Doses (Pfizer)

*Needle and Syringe Sizes for **Pediatric** Ancillary Kits*

PRODUCT	QUANTITY
Needle (25G x 1")	210
Syringe (1 mL, LDV)	210
Needle (25G x 1")	105
Syringe (1 mL)	105
Needles, Mixing (21–25G x 1.5")	60
Syringe, Mixing (3 mL or 5 mL)	60
Alcohol Pad (sterile, individually sealed)	630
Vaccination Record Card	300
Needle Gauge and Length Chart	3
Face Shield	9
Surgical Mask	18
Diluent, 10 mL	30

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccine Supporting 450 Doses (Pfizer-BioNTech)

*Needle and Syringe Sizes for Large Combined Adult Ancillary Kits (currently unavailable)**

PRODUCT	QUANTITY
Needle (22–25G x 1")	315
Syringe (1 mL, LDV)	315
Needle (22–25G x 1")	75
Syringe (1 mL)	75
Needle (22–25G x 1.5")	85
Syringe (1 mL)	85
Needle, Mixing (21–25G x 1.5")	80
Syringe, Mixing (3 mL or 5 mL)	80
Alcohol Pad (sterile, individually sealed)	1,200
Vaccination Record Card	450
Needle Gauge and Length Chart	4
Face Shield	10
Surgical Mask	20
Diluent	75

Needle and Syringe Sizes for Large Combined Pediatric Ancillary Kits

PRODUCT	QUANTITY
Needle (25G x 1")	315
Syringe (1 mL, LDV)	315
Needle (25G x 1")	160
Syringe (1 mL)	160
Needle, Mixing (21–25G x 1.5")	80
Syringe, Mixing (3 mL or 5 mL)	80
Alcohol Pad (sterile, individually sealed)	1,200
Vaccination Record Card	450
Needle Gauge and Length Chart	4
Face Shield	10
Surgical Mask	20
Diluent	75

* Over the course of the program, requests for adult ancillary kits have exceeded anticipated demand while requests for pediatric ancillary kits have been lower than anticipated. As a result, we have exhausted the adult kits and we will ship the pediatric ancillary kit for all Pfizer 450 vaccine orders until the inventory is depleted.

As a reminder, the difference between the Pfizer 450 adult and pediatric ancillary kits is the adult kits include ~20% 1.5" needles used to vaccinate certain adults, based on weight. Pediatric kits include 100% 1" needles. With substitution of a pediatric ancillary kit for an adult ancillary kit, sites should proactively acquire a stock of the 1.5" needles to vaccinate residents/patients/staff who require the longer needle length, as needed. Refer to the Vaccine Administration: Needle Gauge and Length chart included with ancillary kits for information about when the 1.5" needle length should be used.

COVID-19 Vaccine

Product Information Guide



Possible Ancillary Kits—COVID-19 Vaccine Supporting 1,170 Doses (Pfizer-BioNTech)

*Needle and Syringe Sizes for Large Combined **Adult** Ancillary Kits**

PRODUCT	QUANTITY
Needle (22–25G x 1")	830
Syringe (1 mL, LDV)	830
Needle (22–25G x 1")	185
Syringe (1 mL)	185
Needle (22–25G x 1.5")	225
Syringe (1 mL)	225
Needle, Mixing (21–25G x 1.5")	205
Syringe, Mixing (3 mL or 5 mL)	205
Alcohol Pad (sterile, individually sealed)	2,900
Vaccination Record Card	1,200
Needle Gauge and Length Chart	10
Face Shield	25
Surgical Mask	50
Diluent	200

Needle Gauge and Length Chart: www.cdc.gov/vaccines/hcp/admin/downloads/vaccine-administration-needle-length.pdf

* Some needles from the adult kit may be used for adolescents age 12 through 18 years. Choose the correct needle gauge and needle length using CDC's Vaccine Administration: Needle Gauge and Length job aid (<https://www.cdc.gov/vaccines/hcp/admin/downloads/vaccine-administration-needle-length.pdf>). For adolescents age 12–18 years, use the 22–25 gauge, 1-inch needles from the ancillary kit. For adults age 19 years and older, see the chart referenced previously to determine the correct size based on sex and weight.

General Needle and Syringe Resources



Web Resources:

- [Vaccine Administration Resource Library](#): Includes resources for preparing, administering, and documenting vaccines



PDF Resources:

- [Vaccine Administration: Needle Gauge and Length–PDF](#)
- [Vaccine Administration: Intramuscular \(IM\) Injection Children 7 through 18 years of age–PDF](#)
- [Vaccine Administration: Intramuscular \(IM\) Injection Adults 19 years of age and older–PDF](#)



Video Resources:

- [Intramuscular \(IM\) Injection: Supplies \(Children Birth through 18 Years of Age\)](#): This training addresses how to select the equipment needed to prepare an intramuscular (IM) injection for children from birth through 18 years of age.
- [Intramuscular \(IM\) Injection: Supplies \(Adults 19 Years of Age and Older\)](#): This training addresses how to select the equipment needed to prepare an intramuscular (IM) injection for adults 19 years of age and older.
- [Intramuscular \(IM\) Injection Sites](#): This training helps providers identify intramuscular (IM) injection sites. A needle is used to inject the vaccine into the muscle.



Training Resources:






- [Vaccine Administration e-Learn](#): A self-paced vaccine administration course that provides comprehensive training using videos, job aids, and other resources

Specific Ancillary Kit Products and Product Information

The following tables, organized by manufacturer or broker, list possible ancillary kit products with an example image (colors may vary) and links to more information. Resources for additional information include:

- **Web** resources
- **PDF** resources
- **Video** resources

Becton Dickinson Products (Phone: 844-823-5433)

ITEM	WHERE TO FIND MORE INFORMATION
<p>Hypodermic Needle PrecisionGlide™ Conventional*</p> <ul style="list-style-type: none"> ■ Gauge: 22-25/Length: 1.5" 	<p>PrecisionGlide™ Conventional Needle overview</p> <p>Visit Becton Dickinson catalog and search product numbers: 305127, 305156, 305194</p>
<p>Safety Hypodermic Needle BD Eclipse™</p> <ul style="list-style-type: none"> ■ Gauge: 22-25/Length: 1-1.5" ■ Gauge: 23/25/Length: 1-1.25" 	<p>BD Eclipse™ Needle overview</p> <p>Visit Becton Dickinson catalog and search product numbers: 22-25G, 1-1.5": 305762, 305761, 305763, 305767 23/25, 1-1.25": 305866, 305891, 305892</p> <p>BD Eclipse™ directions for use</p> <p>BD Eclipse™ Needle instruction video</p>
<p>Safety Combo, Syringe with BD Eclipse™ Needle 3mL BD Luer-Lok™ Syringe</p> <ul style="list-style-type: none"> ■ Gauge: 22-25/ Length: 1-1.5" 	<p>BD Eclipse™ Needle overview</p> <p>Visit Becton Dickinson catalog and search product numbers: 305782, 305783, 305787</p> <p>BD Eclipse™ directions for use</p> <p>BD Eclipse™ Needle Instruction video</p>
<p>1mL/3mL BD Luer-Lok™ Syringe</p> 	<p>Conventional syringe overview</p> <p>Visit Becton Dickinson catalog and search product numbers: 309628, 309657</p>
<p>Conventional Combo*</p> <ul style="list-style-type: none"> ■ Syringe (3mL/5mL) with attached needle Gauge: 22-25/Length: 1-1.5" 	<p>Conventional syringe overview</p> <p>Visit Becton Dickinson catalog and search product numbers: 309361, 309571, 309572, 309574, 309581, 309582, 309589</p>

*For mixing ONLY and NOT for vaccine administration

Becton Dickinson Product catalog: <http://catalog.bd.com/>

PrecisionGlide™ Conventional Needle overview: <https://www.bd.com/en-us/offering/capabilities/syringes-and-needles/conventional-syringes-and-needles/conventional-needles>

BD Eclipse™ Needle overview: <https://www.bd.com/en-us/offering/capabilities/syringes-and-needles/safety-syringes-and-needles/safety-needles/bd-eclipse-needle>

BD Eclipse™ directions for use: https://www.bd.com/documents/guides/directions-for-use/MPS_HY_Eclipse-needle-usage-guidelines_DF_EN.pdf

BD Eclipse™ Needle instruction video: <https://www.bd.com/en-us/company/video-gallery?video=618677712001>

Conventional Syringe overview: <https://www.bd.com/en-us/offering/capabilities/syringes-and-needles/conventional-syringes-and-needles/conventional-syringes>

COVID-19 Vaccine

Product Information Guide



Cardinal Health (Phone: 1-800-964-5227)

ITEM	WHERE TO FIND MORE INFORMATION
Tuberculin Syringe Monoject™ ■ 1 mL Luer Lock Tip Conventional Syringe 	1 mL Luer Lock Syringes product page Visit Cardinal Health webpage and search product numbers: 1180100777, 1180300777
Magellan™ 3mL Syringe with Hypodermic Safety Needle ■ Gauge: 23/25/Length: 1" 	Magellan™ Safety Needle and Syringe Combination product page Visit Cardinal Health webpage and search product numbers: 8881833310, 8881833510
Magellan™ Hypodermic Safety Needle ■ Gauge: 23/25 Length: 1" 	Magellan™ Hypodermic Safety Needle Product page Visit Cardinal Health webpage and search product numbers: 8881850310, 8881850510



Cardinal Health webpage: <https://www.cardinalhealth.com/en.html>

1 mL Luer Lock Syringes product page: <https://www.cardinalhealth.com/en/product-solutions/medical/patient-care/sharp-safety/needles-and-syringes/1-ml-luer-lock-syringes.html>

Magellan™ Safety Needle and Syringe Combination product page: <https://www.cardinalhealth.com/en/product-solutions/medical/patient-care/sharp-safety/needles-and-syringes/safety-needles-and-syringes/magellan-safety-needle-and-syringe-combination.html>

Magellan™ Hypodermic Safety Needle product page: <https://www.cardinalhealth.com/en/product-solutions/medical/patient-care/sharp-safety/needles-and-syringes/safety-needles-and-syringes/magellan-safety-needles.html>

Duopross Meditech Corp (Phone: 1-800-844-1350)

ITEM	WHERE TO FIND MORE INFORMATION
Safety Combo, 1 mL Syringe with Safety Needle* ■ Gauge: 23- 25/Length: 1-1.5" 	Website and product information currently unavailable.
Safety Combo, 3mL Syringe with Safety Needle* ■ Gauge: 23-25/Length: 1-1.5" 	Website and product information currently unavailable.

*Image for product is a placeholder stock image only. Product image is not currently available.

COVID-19 Vaccine

Product Information Guide








Goldbelt (Contact information currently unavailable)

ITEM	WHERE TO FIND MORE INFORMATION
Safety Needle* <ul style="list-style-type: none"> Gauge: 23-25/Length: 1-1.5" 	Website and product information currently unavailable.
Conventional Syringe 1mL* 	Website and product information currently unavailable.
Conventional Syringe 3mL* 	Website and product information currently unavailable.
Conventional Syringe 5mL* 	Website and product information currently unavailable.

*Image for product is a placeholder stock image only. Product image is not available currently.

Gold Coast (contact information not currently available)*




ITEM	WHERE TO FIND MORE INFORMATION
Safety Combo, 1mL Syringe with Safety Needle <ul style="list-style-type: none"> Gauge: 23/Length: 1" 	Product Number: Carepoint 35-8204 See page 31 of this guide for additional information.
Safety Combo, 1mL Syringe with Safety Needle <ul style="list-style-type: none"> Gauge: 25/ Length: 1" 	Product Number: Carepoint 35-8203 See page 31 of this guide for additional information.
Safety Combo, FlipLock 3mL Syringe with Safety Needle <ul style="list-style-type: none"> Gauge: 23/Length: 1" 	Product Number: 822331 See page 32-34 of this guide for additional information.
Safety Combo, FlipLock 3mL Syringe with Safety Needle <ul style="list-style-type: none"> Gauge: 25/ Length: 1" 	Product Number: 825231 See page 32-34 of this guide for additional information.
Safety Needle <ul style="list-style-type: none"> Gauge: 25/ Length: 1" Gauge: 23/ Length: 1" 	Product Numbers: 25G x 1": EasyTouch 802501 23G x 1": EasyTouch 812301 See page 32-34 of this guide for additional information.

COVID-19 Vaccine

Product Information Guide




HTL Strefa (Phone: 877-660-1900)

ITEM		WHERE TO FIND MORE INFORMATION
Safety Combo, 1mL Syringe with Safety Needle ■ Gauge: 25/Length: 1"		HTL Strefa home page (product 6054 currently unlisted) See page 35 of this guide for instructions for use.
Safety Combo, 3mL Syringe with Safety Needle ■ Gauge: 23/Length: 1"		HTL Strefa home page (product 6053 currently unlisted) See page 35 of this guide for instructions for use.
Safety Combo, 3mL Syringe with Safety Needle ■ Gauge: 25/Length: 1"		HTL Strefa home page (product 6055 currently unlisted) See page 35 of this guide for instructions for use.






HTL Strefa home page: <https://htl-strefa.com/home-page/>

Marathon/Smiths Medical (Phone: 941-704-7864)

ITEM		WHERE TO FIND MORE INFORMATION
Safety Combo, 3mL Syringe with Safety Needle* ■ Gauge: 22-25/Length: 1-1.5"		Website and product information currently unavailable. See page 33 for Identification and Connection Guide and QR code for a short video describing the connections. Product numbers: 4234, 4236, 423510

*Image for product is a placeholder stock image only. Product image is not available currently.

Medline (Contact information currently unavailable)

ITEM		WHERE TO FIND MORE INFORMATION
Safety Combo, 1mL Syringe with Safety Needle ■ Gauge: 23/Length: 1" ■ Gauge: 25/Length: 1"	 	Website and product information currently unavailable.
Safety Needle ■ Gauge: 23/Length: 1-1.5" ■ Gauge: 25/Length: 1"	 	Website and product information currently unavailable.
Conventional Syringe ■ 1mL ■ 3mL		Luer Lock Syringe product page Visit Medline webpage and search product numbers: SYR101010, SYR103010

Luer Lock Syringe product page: <https://www.medline.com/product/Luer-Lock-Syringes/Syringes-without-Needle/Z05-PF11377?question=SYR103010&index=P1&indexCount=1#mrkOrderingInfoTable>



Medline webpage: <https://www.medline.com>

COVID-19 Vaccine

Product Information Guide




Quality Impact (Contact information currently unavailable)*

ITEM	WHERE TO FIND MORE INFORMATION
Safety Combo, 1mL Syringe with Safety Needle [†] ■ Gauge: 23-25/Size: 1" 	Website and product information currently unavailable.
Safety Combo, 3mL Syringe with Safety Needle [†] ■ Gauge: 23-25/Size: 1" 	Website and product information currently unavailable.

* HHS discontinued use of HAIUO syringes, distributed by Quality Impact, in March 2021 due to feedback received from providers about the poor quality. HHS replaced all of the HAIUO needles/syringes that were reported to HHS and/or McKesson Customer Service.

[†]Image for product is a placeholder stock image only. Product image is not available currently.

[Retractable Technologies Inc.](#) (Phone: 888-703-1010)

ITEM	WHERE TO FIND MORE INFORMATION
Safety Combo Unit, 1mL/3mL, with attached needle ■ Gauge: 23/25/Length: 1" 	Visit Retractable Technologies Inc. webpage and search product numbers: 10161, 10311, 10391 VanishPoint® Syringes brochure VanishPoint® Syringes product usage information VanishPoint® Syringes video

Retractable Technologies Inc. webpage: <https://retractable.com/Products>

VanishPoint® Syringes brochure: <https://d2ghdaxqb194v2.cloudfront.net/577/166728.pdf>

VanishPoint® Syringes product usage information: <https://d2ghdaxqb194v2.cloudfront.net/577/166715.pdf>

VanishPoint® Syringes video: <https://youtu.be/wC-uXq3uUdQ>

COVID-19 Vaccine

Product Information Guide



Gold Coast CarePoint Safety Needles and Syringes

Available at: <http://www.allisonmedical.com/?product=safety-needle> and <http://www.allisonmedical.com/?product=safety-needle-combinations>



SAFETY NEEDLES

25/23 Gauges (Variety of Lengths)



- Boxes of 50, sterile needles are sealed in individual packages for assurance and convenience
- Compatible with all major brands of luer lock syringes
- Lubricated needle provides smooth injection

- Easy, three-way safety wing activation: forefinger, thumb or hard surface
- Needles are UV bonded to ensure stability
- Easy to read tri-lingual packaging

Item Number	Product Description	UPC	Box Quantity	Case Quantity
32-8700	25G x 5/8" (.625)	786227081005	50 ct.	16/bxs
32-8701	25G x 1"	786227081015	50 ct.	16/bxs
32-8702	25G x 1 1/2" (1.5)	786227081025	50 ct.	16/bxs
32-8703	23G x 1"	786227081035	50 ct.	16/bxs
32-8704	23G x 1 1/2" (1.5)	786227081045	50 ct.	16/bxs

LUER LOCK SYRINGES WITH SAFETY NEEDLES

3cc Syringes



- Boxes of 50, sterile needles are sealed in individual packages for assurance and convenience
- Compatible with all major brands of luer lock syringes
- Lubricated needle provides smooth injection

- Easy, three-way safety wing activation: forefinger, thumb or hard surface
- Needles are UV bonded to ensure stability
- Easy to read tri-lingual packaging

Item Number	Product Description	UPC	Box Quantity	Case Quantity
35-8200	25G x 5/8" 3cc	786227082005	50 ct.	8/bxs
35-8201	25G x 1" 3cc	786227082015	50 ct.	8/bxs
35-8202	23G x 1" 3cc	786227082025	50 ct.	8/bxs



info@allisonmedical.com • allison medical • www.allisonmedical.com

inspired | innovation

For questions regarding the CarePoint Safety Line, contact Allison Medical Customer Service at: 1-800-886-1618

Gold Coast Easy Touch FlipLock Safety Syringes Brochure

Easy•Touch® **FlipLock™** Safety Syringes

SIMPLE & SMART

EasyTouch FlipLock Safety Syringes offer an easy and simple solution in caregiver protection. After injection, just flip the protective shield over the needle until it clicks and locks in place. Hands and fingers stay away from the exposed needle. Simple, smart and cost-effective.



- ✓ Easy to Use
- ✓ Protective Cover Shields Needle After Use
- ✓ Familiar Technology Requires Limited Training
- ✓ Multi-lock Safety System
- ✓ Contains No Natural Rubber Latex

- ✓ Heavy-Duty Barrel
- ✓ Disposable
- ✓ Sterile, Individually wrapped
- ✓ Needle Attaches to Any Luer Lock Barrel
- ✓ Safety Engineered

Gold Coast Easy Touch FlipLock Safety Syringes Brochure

Easy•Touch® FlipLock™ Safety Syringes

FlipLock™ Safety Syringe w/ Exchangeable Needle



ITEM#	ML/CC	GAUGE	LENGTH	BOX QTY	CASE QTY	NDC# 08496-
822331	3mL	23G	25mm, 1"	100/box	6/case	0263-01
822337	3mL	23G	40mm, 1.5"	100/box	6/case	0293-01
825231	3mL	25G	25mm, 1"	100/box	6/case	7012-01
825211	1mL	25G	25mm, 1"	100/box	6/case	0248-01

FlipLock™ Needles



ITEM#	GAUGE	LENGTH	BOX QTY	CASE QTY	NDC# 08496-
812301	23G	25mm, 1"	50/box	5/case	0201-01
812307	23G	40mm, 1.5"	50/box	5/case	0202-01
812501	25G	25mm, 1"	50/box	5/case	0204-01

Luer Lock Barrels




ITEM#	ML/CC	DESCRIPTION	BOX QTY	CASE QTY	NDC# 08496-
802015	1mL	Luer Lock Barrel (INS)	100/box	6/case	0164-01
802030	3mL	Luer Lock Barrel (TB)	100/box	6/case	0159-01


Gold Coast Easy Touch Fluringe Fliplock Safety Syringes

Easy•Touch® **FLURINGE®** FLIPLOCK™ SAFETY SYRINGES


- 1 LOAD SYRINGE**
Pull the plunger back to load the syringe

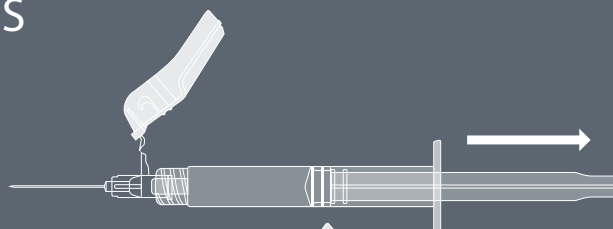

- 2 INJECT**
Push the plunger in one smooth motion to administer medication

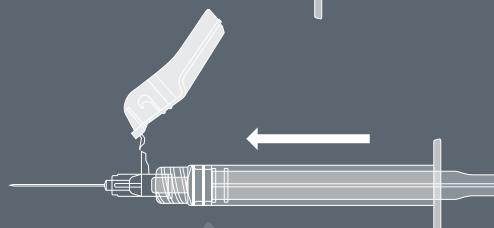

- 3 FLIP**
Flip the protective shield over the used needle until it clicks into place. Discard according to local regulations

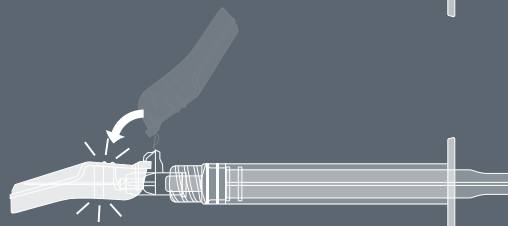

COVID-19 Vaccine

Product Information Guide



HTL Strefa DropSafe Syringe with Safety Needle Instructions for Use

DropSafe

Syringe with Safety Needle

Manufactured for:

HTL-STREFA S.A.
ul. Adamówek 7, 95-035 Ozorków, Poland
T: +48 42 270 00 10; F: +48 42 270 00 20
www.htl-strefa.com, E: info@htl-strefa.pl

Distributed by:

HTL-STREFA, Inc.
3005 Chastain Meadows Pkwy, Suite 300
Marietta, GA 30066, USA
T: +1 770 528 0410; F: +1 770 528 0411
www.htl-strefa.com; E: info@htl-strefa.com



The Sterile Disposable Syringe with Safety Needle is intended for use in the aspiration and injection of fluids for medical purpose.

Ask your healthcare professional for assistance in choosing the needle length, injection site and technique appropriate for you.


The needle fits any standard luer lock or luer slip syringe and it is equipped with a safety cover, that prevents needle stick self-injury after use.

INSTRUCTIONS FOR USE


Follow your recommended clinical procedures for drug injections. See the patient information leaflet of the drug.

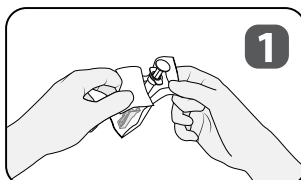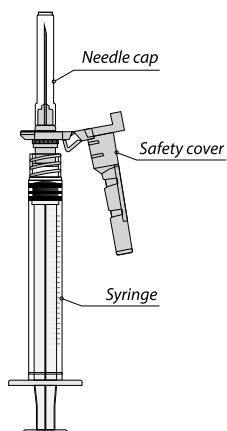


Peel blister pack of the DropSafe Syringe with Safety Needle. Use aseptic technique.

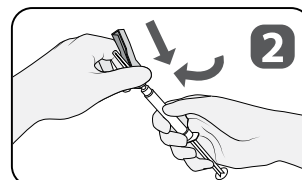


Ensure needle is firmly attached to the luer lock or luer slip syringe.

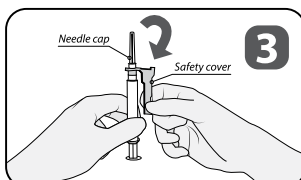


Pull back the safety cover to enable a clear view of the injection site then remove the needle cap.

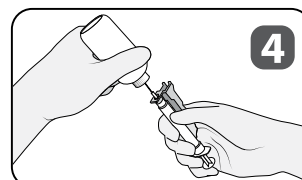


Draw up the medication according to its reference information. Prime the needle, if required.

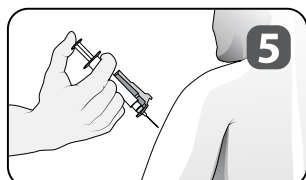

Administer injection according to established protocol.

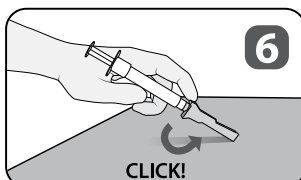

Do not recap needle with needle cap. Immediately after injection, activate the safety cover by pressing down on a flat surface until you feel or hear an audible click, indicating the cover is securely locked over the needle.

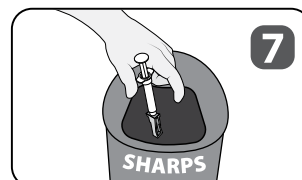

Once the cover is securely locked over the needle, dispose of syringe and needle in an approved sharps container.

Retractable Technologies Inc VanishPoint® Syringe Flyer


Available at: <https://d2ghdaxqb194v2.cloudfront.net/577/166715.pdf>




VANISHpoint®

Syringe


Standard Draw Procedure

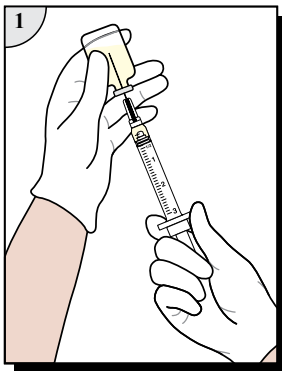

In-Patient, Automated Retraction

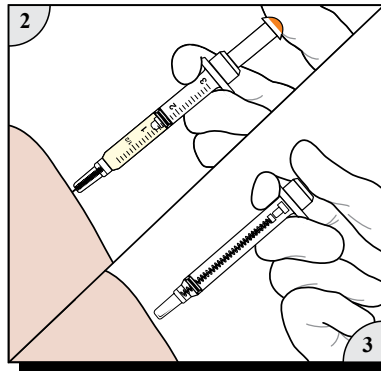

Non-Reusable

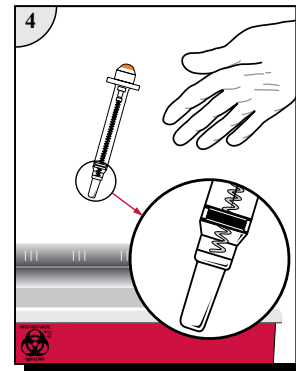

Product Usage Information:

1. Prepare and give injection using aseptic technique according to institutional policy.
2. For injection into patients, continue depressing plunger to activate automatic needle retraction **while needle is still in patient**. For injection into IV ports, continue depressing plunger to activate automatic needle retraction and **immediately remove needle from port**. **Full dose is administered only when needle retraction is activated**.
3. Needle will automatically retract into syringe, preventing exposure to contaminated needle and rendering syringe non-reusable. In the event that needle retraction mechanism does not activate, discard syringe in an appropriate sharps container per protocol of institution. Do not recap contaminated needles.
4. Dispose of VanishPoint® syringe in an appropriate sharps container per protocol of institution.

Precautions:

- Single use only. Reuse of this device may result in exposure to bloodborne pathogens, including Hepatitis B virus (HBV), Hepatitis C virus (HCV), and human immunodeficiency virus (HIV).
- Contents are sterile, non-toxic, and non-pyrogenic. Do not use if product or package is damaged.
- Not made with natural rubber latex.
- Use only with attached needle. Needle cannot be changed.
- Automated needle retraction occurs only when barrel is emptied and plunger is fully depressed.
- For applications where full dose is not administered, expel remaining contents according to institutional policy and activate needle retraction.
- U.S. Federal Law restricts this device to sale by or on the order of a physician.

Smiths Medical Needle-Pro® and Needle-Pro® Edge™ Safety Devices

Needle-Pro® and Needle-Pro® Edge™ Safety Devices

Identification and Connection Guide

Smiths Medical offers the Needle-Pro® and Needle-Pro® Edge™ hypodermic safety devices. It is important to identify which device you are using and review the Instructions for Use prior to using. The information below is to assist you in identifying which device you are using and to help ensure the connections are tightened and secure prior to use.

Needle-Pro® Hypodermic Safety Device




Two points of connection

- Needle to Needle-Pro® orange safety device [1]
- Needle-Pro® orange safety device to syringe [2]

Ensure the 2 connections are secure by:

1. Twisting to tighten the needle into the Needle-Pro® orange safety device AND
2. Twisting to tighten the Needle-Pro® orange safety device into the syringe

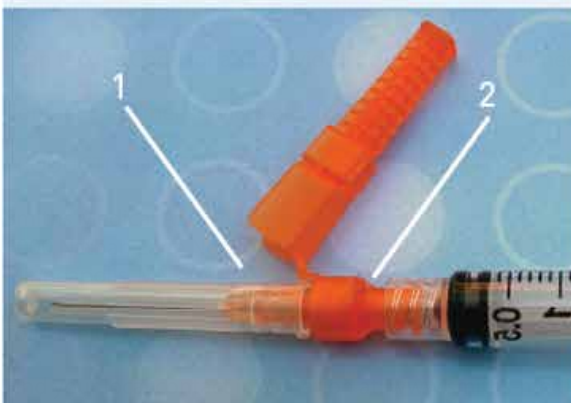

Needle-Pro® EDGE™ Hypodermic Safety Device



One connection

- Needle-Pro® Edge™ safety device to syringe [1]

Ensure the connection is secure by:

1. Twisting to tighten the connection between the Needle-Pro® Edge™ and syringe prior to use



Scan the QR Code for a short video describing the connections.



Contact customer services at 1-800-258-5361 or at www.smiths-medical.custhelp.com

PRODUCT(S) DESCRIBED MAY NOT BE LICENSED OR AVAILABLE FOR SALE IN CANADA AND OTHER COUNTRIES

Smiths Medical ASD, Inc.
6000 Nathan Lane North
Minneapolis, MN 55442, USA
Phone: 1-214-618-0218
Toll-Free USA 1-800-258-5361

Find your local contact information at: www.smiths-medical.com/customer-support

Smiths Medical is part of the global technology business Smiths Group plc. Please see the Instructions for Use/Operator's Manual for a complete listing of the Indications, contraindications, warnings and precautions. Jelco, NeedlePro, and the Smiths Medical design mark are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. All other names and marks mentioned are the trademarks or service marks of their respective owners. ©2021 Smiths Medical. All rights reserved. VA0241.EN.AM.Rev.A.0121

**Rx
ONLY**
smiths medical

COVID-19 Vaccine

Product Information Guide



Pfizer-BioNTech COVID-19 Vaccine and Low Dead-Volume (LDV) Syringes and/or Needles

Available at: <https://www.cvdvaccine-us.com/images/pdf/Low-Dead-Volume-Syringe-Brochure.pdf>

Pfizer-BioNTech COVID-19 Vaccine and LOW DEAD-VOLUME (LDV) SYRINGES AND/OR NEEDLES

The Pfizer-BioNTech COVID-19 Vaccine has not been approved or licensed by FDA, but has been authorized for emergency use under an Emergency Use Authorization to prevent Coronavirus Disease 2019 (COVID-19) for use in individuals 16 years of age and older. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of the medical product under Section 564(b)(1) of the FD&C Act unless the declaration is terminated or authorization revoked sooner.


Please see Emergency Use Authorization (EUA) Fact Sheet for Healthcare Providers Administering Vaccine (Vaccination Providers) including Full EUA Prescribing Information available at www.cvdvaccine-us.com.

After dilution, vials of Pfizer-BioNTech COVID-19 Vaccine contain 6 doses of 0.3 mL of vaccine. Low dead-volume syringes and/or needles can be used to extract 6 doses from a single vial.

Use of low dead-volume (LDV) syringes and/or needles may maximize the potential number of vaccine doses and minimize vaccine wastage¹

i Low dead-volume (also called low dead-space) is the amount of fluid remaining within the syringe and needle after an injection is completed²

Characteristics of LDV syringes and needles

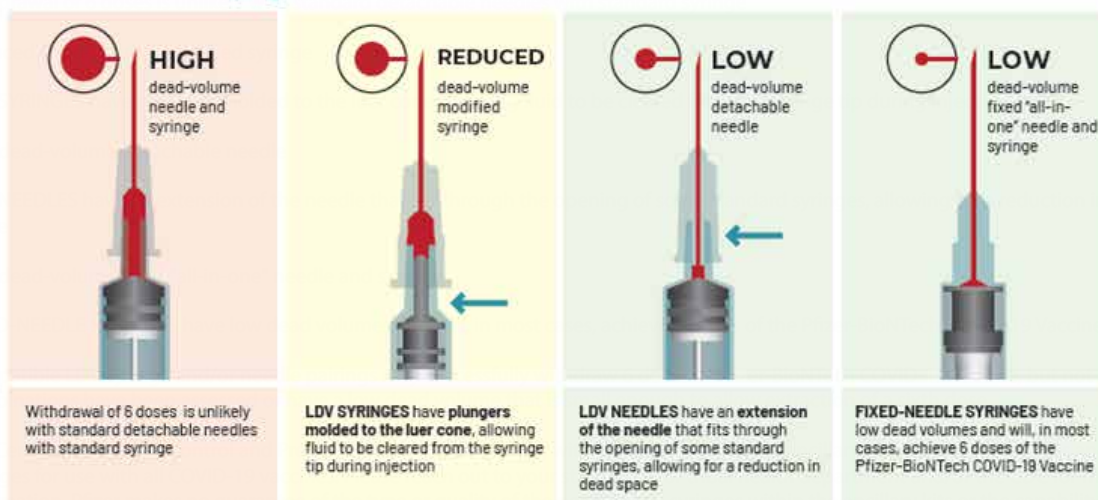

Compatible LDV syringe and/or needle pairings may successfully withdraw 6 doses of Pfizer-BioNTech COVID-19 Vaccine, but not all combinations have been assessed or have a low dead volume that is small enough to allow extraction of a sixth dose.

The Centers for Disease Control and Prevention (CDC) is partnering with McKesson to provide ancillary kits containing needles and syringes for use with all COVID-19 vaccines. Please reach out to your state, local, or tribal health department and/or the CDC for more information. Pfizer makes no guarantee of the type or quality of the needles and syringes within the ancillary kits or the capability to withdraw 6 doses. For further information, please contact the manufacturer directly.

Find out more about the Pfizer-BioNTech COVID-19 Vaccine at

www.cvdvaccine-us.com



References: 1. Jarrahan C, Rein-Weston A, Saxon G, et al. Vial usage, device dead space, vaccine wastage, and dose accuracy of intradermal delivery devices for inactivated poliovirus vaccine (IPV). *Vaccine*. 2017;35:1789-1796. 2. Kesten JM, Ayres R, Neale J, et al. Acceptability of low dead space syringes and implications for their introduction: a qualitative study in the West of England. *Int J Drug Policy*. 2017;39:99-108.



Manufactured by
Pfizer Inc.
New York, NY 10027



Manufactured for
BioNTech Manufacturing GmbH
An der Goldgrube 12
55131 Mainz, Germany

PF-CV19-USA-0180
© 2021 Pfizer Inc. All rights reserved. February 2021